


TRUS LIFT

OWNERS MANUAL Copyright 1996

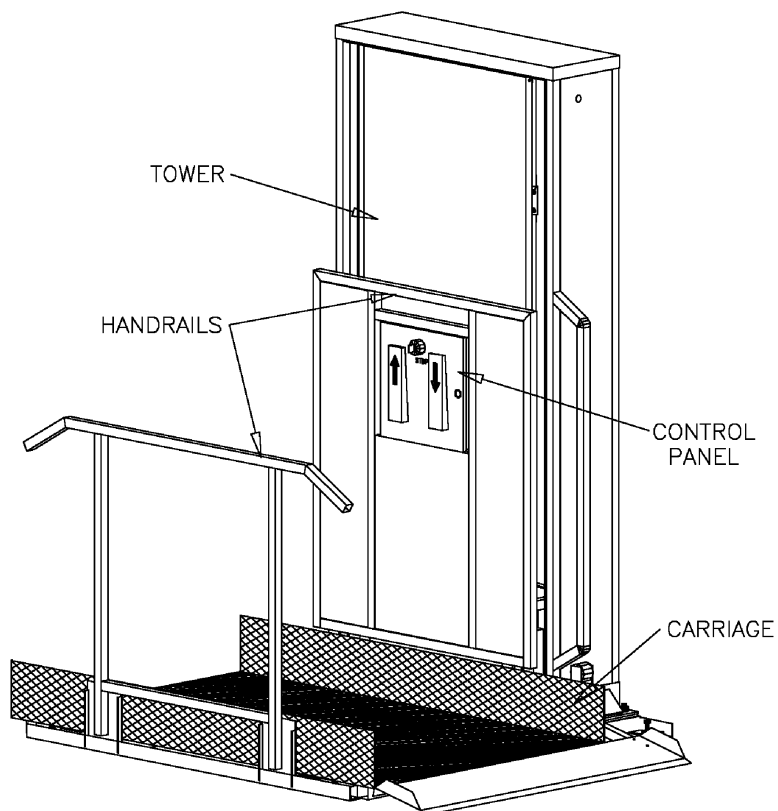

Thank you for selecting the Trus<T>Lift. The Trus<T>Lift when operated properly is designed to provide years of trouble free service. This manual is provided to show you how to operate the lift safely and efficiently. Please read this manual thoroughly before operating the lift for the first time.

SAFETY PRECAUTIONS

- Make sure that there is nothing obstructing the carriage travel before operating the lift.
- The lift is intended for use by one passenger and one wheelchair only, to a maximum of 550 lb. **(DO NOT OVERLOAD THE LIFT)**.
- Make sure that the passenger and the wheelchair are completely on the carriage before raising or lowering the lift.
- Please use the handrails while the carriage is in motion (See Cover Figure).

OPERATING INSTRUCTIONS

- To raise or lower the carriage press either the UP or DOWN paddle (See Figure 1) and maintain continuous pressure on the paddle until the lift stops automatically at the landing.
- If you decide to change the direction of travel of the lift before it reaches the landing, you must release the paddle and wait for the lift to come to a COMPLETE STOP before pressing the other paddle.

If you need to stop the lift before it reaches the landing, simply release the UP or DOWN paddle and the carriage will stop immediately. You can also stop the lift by pressing the red STOP button (See Figure 1). To release, simply pull out the button.

A key switch is located near the UP and DOWN paddles to prevent unauthorized use of the Trus<T>Lift. The key is removable in both positions. The switch may be turned off and the key removed making the unit inoperable. When you wish to use the lift again insert the key, turn the switch on, and remove the key. It is recommended that you have the key readily available whenever operating the lift.

MANUAL CRANK OPERATION

If a power outage occurs while the lift is in use and a passenger is stranded between landings, the carriage can be manually raised or lowered using the Hand Crank Tool provided:

- Unplug the power cord.
- Insert the square end of the Hand Crank Tool through the access hole at the upper right side of the tower (See Figure 2).
- Turn the Hand Crank Tool clockwise to lower the lift and counter-clockwise to raise the lift.

REPLACING FUSES

- Unplug the power cord.
- Open the control box with the key provided (this is the key numbered 644).
- The fuses are located in the control box on the right hand side (See Figure 3).
- Replace the blown fuse with a fuse with the same Amp rating.

For maintenance and trouble shooting information, please refer to the Trus<T>Lift Installation Manual included with the lift or visit our website www.trustram.com

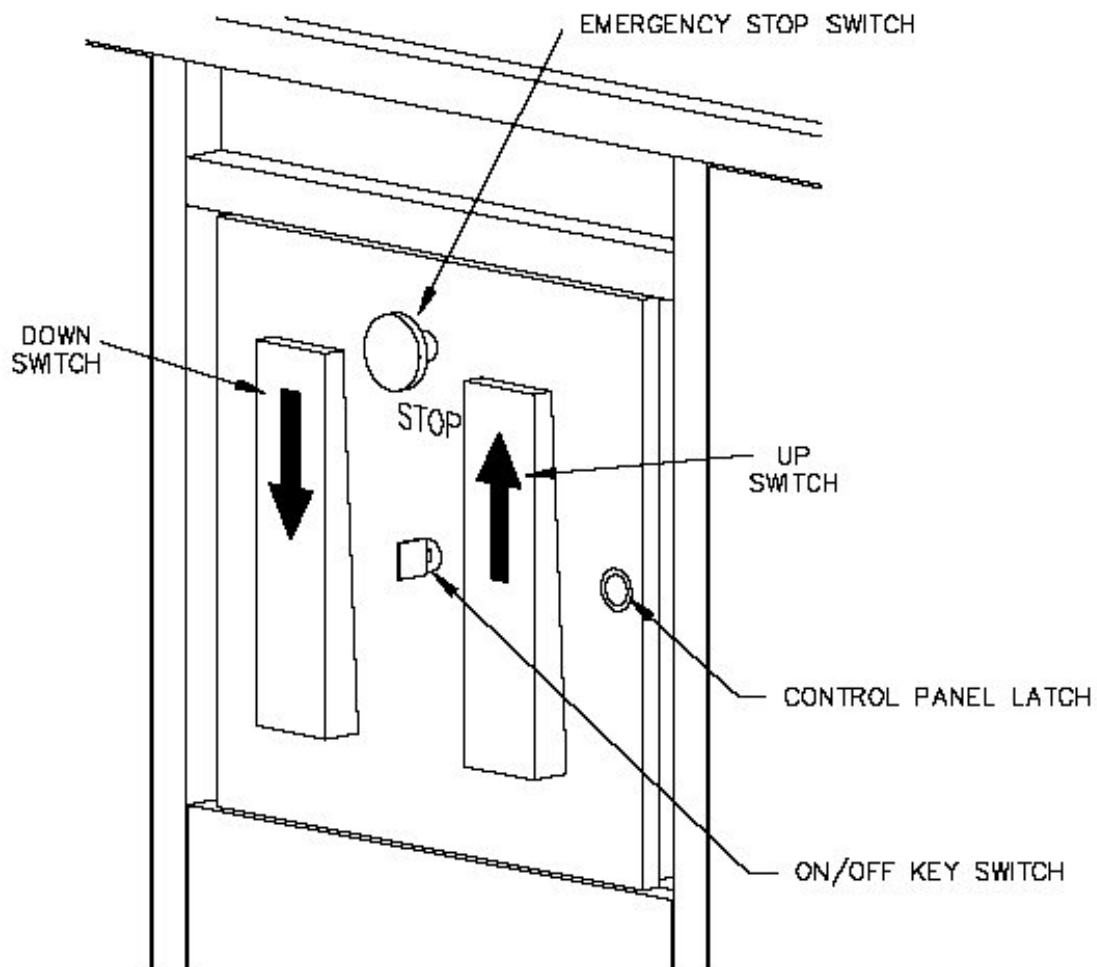


Figure 1. Location of UP, DOWN, and STOP controls.

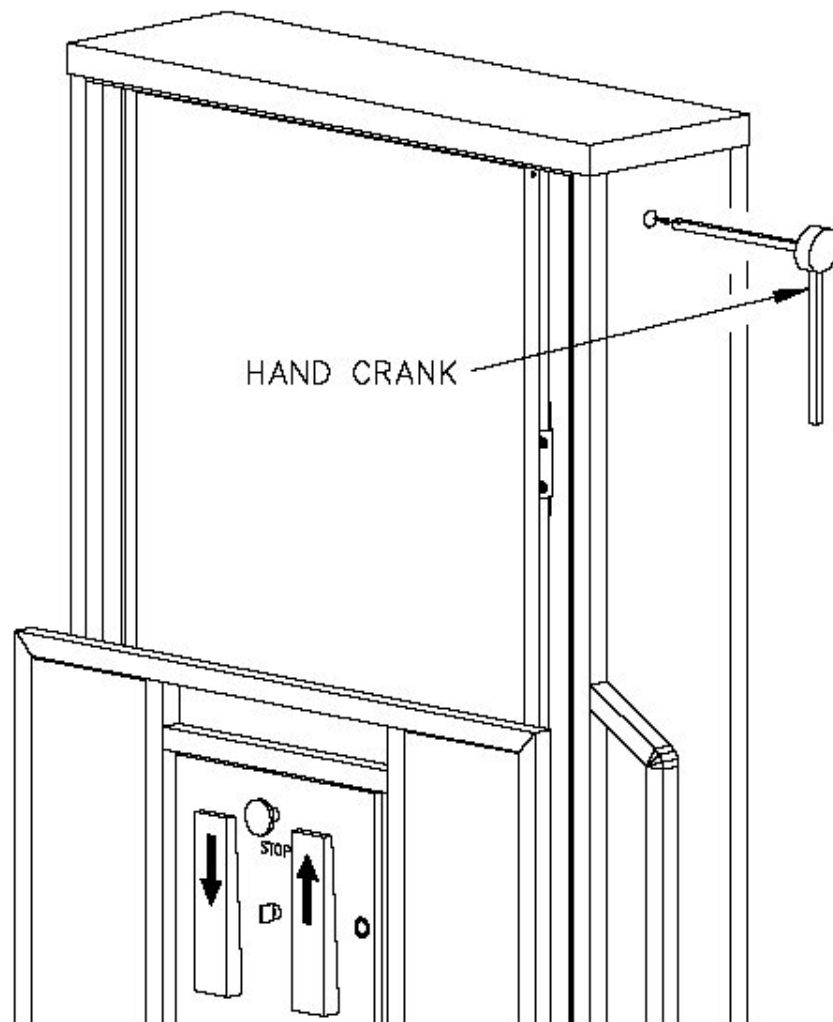


Figure 2. Manual crank.

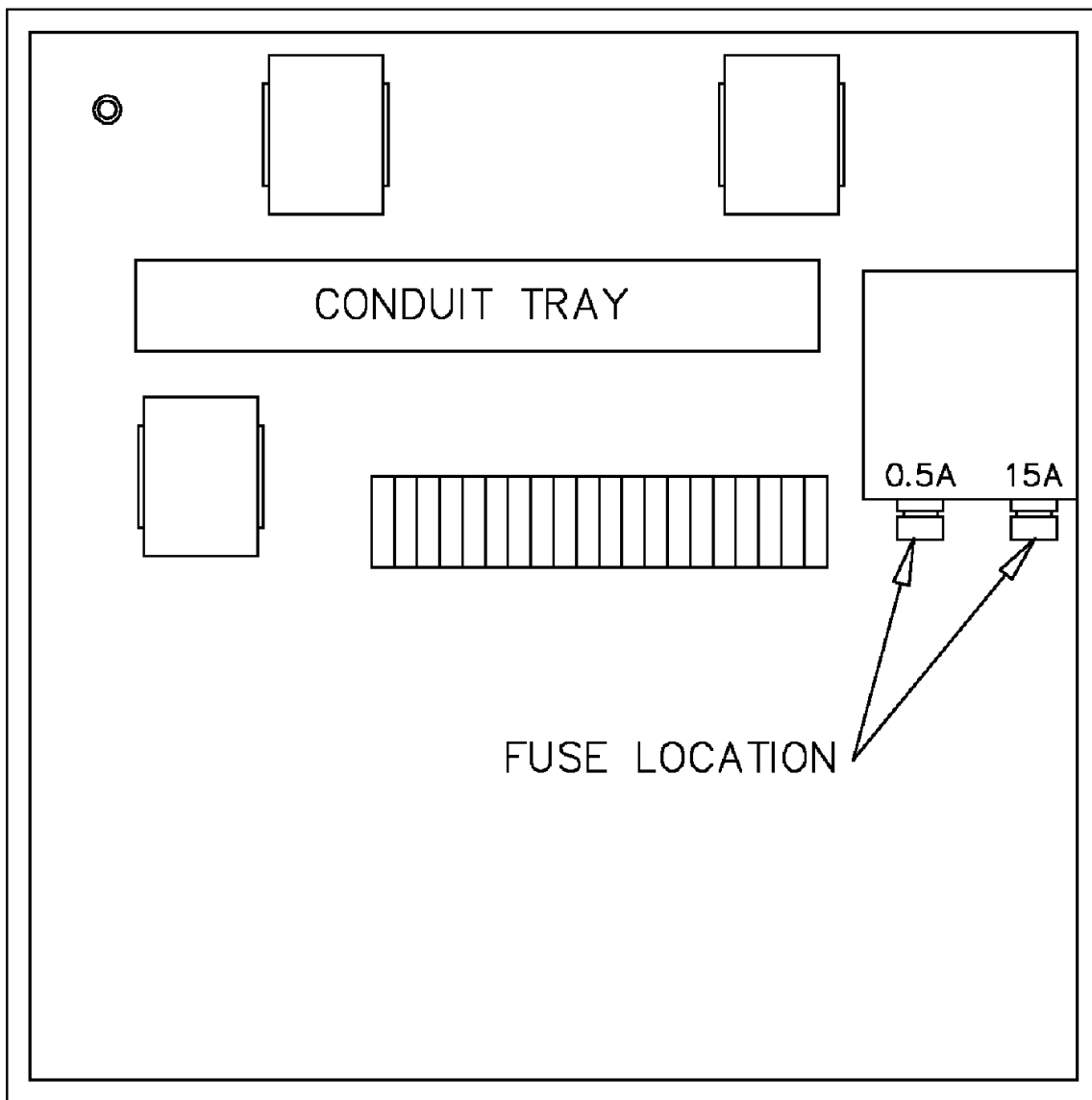


Figure 3. Fuse Location.

